

Technical Sheet

Rev. 02 of 06/02/2020 - Mod. 73-A8_EN


CS PLUS 10



GENERAL DATA

Disposable container for sharp and cutting objects. Item designed for frequent use (Hospitals, Healthcare Centres, Clinics)

- ▶ **Material:** Plastic material 100 % virgin Polypropylene
- ▶ **Colour:** Yellow body and red lid in order to signal the dangerous content. Materials coloured with masterbatch without heavy metals and cadmium. Materials compliant to REACH
- ▶ **Shape:** Square base
- ▶ **Weight:** 594 gr.
- ▶ **Max Load:** 5000 gr.
- ▶ **Nominal capacity:** 10,00 Lt.
- ▶ **Real capacity:** 9,35 Lt.
- ▶ **Useful capacity:** 7,90 Lt
- ▶ **External dimensions:** 220x220x325 H mm (lid assembled)

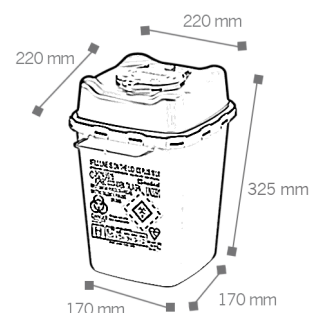

LOGISTIC DATA:

- ▶ **Pcs per carton:** 33 (body and lid not assembled)
- ▶ **Carton dimensions:** 79x60x48 H cm
- ▶ **Pcs per pallet:** 264 pcs.
- ▶ **Pallet dimensions:** 120x80x204 h cm

CERTIFICATIONS:

- ▶ Compliant to AFNOR NF X-30-511
- ▶ Compliant to NF Mark
- ▶ Compliant to ADR -UN approved
- ▶ Compliant to ISO 23907-1 - Kite Mark
- ▶ Compliant to TRBA 250

INSTRUCTIONS

► Lid Assembly:

Place the opening of the lid above the label and press firmly on the opposite corners clicking all the joints and after on the opposite sides.



► Temporary closure:

Lower the lid holding the tongue on «1PROV», then push the tongue to block the edge

► At this stage closure is temporary



► Permanent closure:

Place the lid in temporary closure and press firmly on the two central «1 DEF» and after on the «2 DEF». At this stage, closure is permanent.



► Needles removal:

Use the «V» removal device to separate needles and syringes

Use the «marguerite» device to unscrew the insulin pen



► **Holders:**

Wall trolley bracket with grabs



SAFETY INFORMATION:

- Use the product only with the appropriate lid
- Do not use the product before fitting the lid
- Do not attempt to rescue objects already in the container
- Respect the filling level indicated: max. 85 % of real capacity
- Do not try to open the container once permanently and irreversibly sealed
- Container to dispose according to the existing law
- Follow the instructions of use to ensure the safety of the container
- Use handles to move the container

ADVANTAGES:

- Lid provided with safe closure device, permanent closure
- Easy to use and to handle
- Sharps and cutting objects resistant
- Handles available
- Lid provided with needles removal devices
- Suitable to autoclave (compliant to NFX 30-511 even after sterilization)
- Maximum filling level indicated
- Several holders available
- Irreversible permanent closure